



Synapse 4.2.200 introduces a fix for the following issues:

- REF# 28892: Incorrect Mean and Standard Deviation After Applying Different Presets on the Same Image in Two View Ports
- REF# 28823: Slow Scrolling Performance on Calibrated 4 + 1 Monitor Configuration with Aux Window Enabled
- REF# 28741: Lost Annotation While Performing Drag/Drop After Creating Annotations and Applying Preset
- REF# 28644: Loss of Focus when Encountering Blank or Missing Series View Port
- REF# 28513: Folder Caching Tool Has Synonym-Related Issues
- REF# 28471: Poor Performance when Synapse DicomSCU Displays Images from GE PACS
- REF# 28454: Different Measurements Appear when Switching Between View States
- REF# 28418: When Report Impression Is On, Synapse Shows Different Number of Studies Returned between PowerJacket and All Studies
- REF# 28391: Annotations Added for Images Are Lost in Certain Situations
- REF# 28373: Series Reallocation Does Not Update Study Image Count Correctly
- REF# 28370: Intermittent Script Error Occurs when Opening a Study
- REF# 28331: Windows Explorer Crashes when Turning on Auto-Compare on Mammo Workstations
- REF# 28239: Subscription Settings Turned Off when Closing Settings Folder Opened from a Folder List
- REF# 28211: Drag-and-Drop Annotations on MG Images Are Lost after Being Overwritten by Other Images via Drag-and-Drop
- REF# 28202: Opening Reading Protocol Editor when Monitors Are Not Top-Aligned Incurs Script Error
- REF# 28197: Image Processing Is Not Applied to "For Processing" SOP Classes (MG and DX) when Retrieving Image via WADO URL
- REF# 28131: "Import All" Button in "Batch Import Reading Protocol" Dialog Box Cannot Be Enabled
- REF# 27918: MG Stack "Conversion All Priors" Option Not Available for Non-Autogen P1 Study
- REF# 27677: Icons for Synapse Toolbar Buttons Misplaced for Users Without Dictation Rights
- REF# 27648: Annotations Not Saving after Window Level Change, Drag/Copy, and Applying 'Save Image' when Selecting 'No' to CTRL + F5
- REF# 27406: SR-1000085 - Procedure Code Consolidation Tool Fails
- REF# 27221: Dragging & Dropping Protocols Between Folders Causes Protocol Steps to Be Rearranged
- REF# 27677: Icons for Synapse Toolbar Buttons Misplaced for Users Without Dictation Rights
- REF# 27572: The Character Set Value (0008, 0005) Is Not Placed in the KOS Generated by Manual Publishing
- REF# 27547: New DICOM Toolkit to Address Improper BEE UT VR Encoding Is Available
- REF# 27439: XDS Consumer Shows Script Error when Retrieved KOS Manifest Contains Specific Character Set
- REF# 27320: Crash at fujifld.dll for Rundll32 Process when Trying to Calibrate Monitor
- REF# 27241: Synapse 4.1 - Clicking on the Hyperlink of a Data Source Name on the Data Sources Page in eSWAT Causes IE to Be Unresponsive
- REF# 27106: SWAT:Admin Report: Does Not Produce the Right Information Based on the Custom Folder Filter Criteria
- REF# 27062: Synapse 4.1.300 - DICOM Server Localization Does Not Work for MWL C-Find Response: Invalid Character Set
- REF# 26892: Synapse 4.1.300 - When Making a Custom Folder from Reporting Events with a Filter for 'report status' or 'report status code', the Folder Yields No Results
- REF# 26795: "Bits Allocated" in LUT Output Is Not Calculated Correctly
- REF# 26700: Synapse 4.1 - Slow Performance for a Custom All Recent Studies Folder
- REF# 26429: Poor Performance for Synapse Designer
- REF# 25621: Synapse 4.1 Hangs when Bringing Up DicomTransfer Dialog Box